# I-M-O-N METHODOS

EL

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EN

Lab Test I-M-O-N KIT

## Dry Blood Spot (DBS) Sampling Instructions

### CONTENT

- Aluminum foil bag containing the sampling card (a). 
- One sampling card with 2 sampling wells (b). 
- Disposable lancet (c). 
- Adhesive bandage (d). 
- Information form & package kit (e). 

### STEP BY STEP INSTRUCTIONS:

- 1.** Wash your hands meticulously with warm water and soap.

- 2.** Sit down comfortably in a chair, with your hands down. Shake them slightly in order to help better blood circulation.

- 3.** Open carefully the aluminum foil bag (a) and remove the CapitainerB sampling device (b). Open the lid of the card and place it down on the table in front of you. Please do NOT touch the sampling wells of the card.

- 4.** Remove the protective cover of the disposable lancet (c). Choose either middle or ring finger to prick with the lancet.


- 5.** Press your finger with your thumb to help your fingertip be filled with blood. Carefully press the lancet against your fingertip to activate the lancet.

- 6.** Release the applied pressure and wait for the blood to appear. Wipe off the first drop.


- 7.** Let a NEW, LARGE drop of blood to form.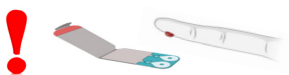

Wait for the drop to fall of due to gravity in to the one sampling well (b).

Apply only ONE drop!

- 8.** Repeat step 7, and fill the second sampling well (b).

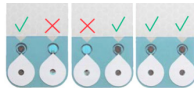- 9.** Wipe the blood with a tissue and apply the adhesive bandage (d) in the fingertip.

- 10.** Please check if the sampling was completed successfully : A) The control window (arrow) should be filled with blood. B) Check that the two sampling wells contain blood, by looking the back-cover of the card. If so, close the lid of the card.

- 11.** Please fill all the necessary information in the enclosed form (e) and return the card to the laboratory.

- USE THE SMALL BOX** of your KIT to package your Sampling specimens and send it back to our Lab.

### ANALYSIS LAB

Tsimiski 51, PC: 54623,

THESSALONIKI, GREECE

Tel: +30 2310 256388, +30 2310 227089

E-mail: [analysis@analysis.gr](mailto:analysis@analysis.gr)

### WAYS OF SENDING YOUR SAMPLING

- As soon as you execute correctly the whole procedure, come in contact with us- STRICTLY within 24h - in order to sent our courier at the time and place best suits you to collect your specimens.

### SEND US using one of the following

E-Mail : [info@ismadeofnature.com](mailto:info@ismadeofnature.com)

Viber : +306934831964

What's up : +306934831964

Your : Name, Address, Mobile Phone and also the day and hour best suits you to book an appointment with our courier partner

**BE SURE WE HAVE RECEIVE YOUR DETAILS BY RESPONDING TO YOUR MESSAGE**

Thank you!

I-M-O-N  
METHODOS

<https://capitainer.com/how-to-use/>  
<https://imonmed.com/services/labtests/>