I-M-O-N METHODOS

EL

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EN

Lab Test I-M-O-N KIT

Buccal Swab DNA Sampling Instructions

CONTENT


- 2 Sampling buccal swab cotton buds in tubes (a) & (b).

- Information form (c)

- package kit (d).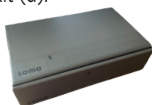

STEP BY STEP INSTRUCTIONS:

- 1 a)** Take your DNA sample at least one hour after eating, drinking, smoking, chewing gum, brushing your teeth, or using mouthwash.
b) Rinse the mouth with warm water ten minutes before swabbing

- 2** Wash your hands meticulously with warm water and soap.

- 3** Pull to open the tube with the sterilized buccal cotton swabs (a) & (b). Remove the buccal cotton buds from the tube, taking care not to touch the white swab head with your fingers

- 4** Carefully insert the cotton head of the swab to your cheek. Place the swab tip in your mouth and rub the side of your inner cheek - up and down- about 6 times and for up to one minute.

- ! **ATTENTION!!! Scrape vigorously, but not to the point that it hurts.**

- 5** Once finished, be sure not to touch the end of the swab to any surface other than the inside of your mouth and the inside of the special plastic tube. Place the swab tip into the opened tube. Put the cap back on the tube.

- 6** To improve the amount of DNA retrieved, take an extra sample from a different side of the mouth or take it several hours apart.

- 7** Please fill in all the necessary information in the enclosed form (c). Put the form inside the packaging box and return it to the laboratory along with the DNA specimen tubes

- USE THE SMALL BOX of your KIT** to package your Sampling specimens and send it back to our Lab.

ANALYSIS LAB
Tsimiski 51, PC: 54623,
THESSALONIKI, GREECE
Tel: +30 2310 256388, +30 2310 227089
E-mail: analysi@analysi.gr

WAYS OF SENDING YOUR SAMPLING

- As soon as you execute correctly the whole procedure, come in contact with us- STRICTLY within 24h - in order to sent our courier at the time and place best suits you to collect your specimens.

SEND US using one of the following
E-Mail : info@ismadeofnature.com
Viber : +306934831964
What's up : +306934831964

Your : Name, Address, Mobile Phone and also the day and hour best suits you to book an appointment with our courier partner

BE SURE WE HAVE RECEIVE YOUR DETAILS BY RESPONDING TO YOUR MESSAGE

Thank you!

I-M-O-N
METHODOS

https://www.youtube.com/watch?v=IUC_xF-yV8
<https://imonmed.com/services/labtests/>