I-M-O-N METHODOS

EL

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
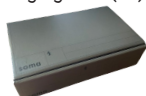
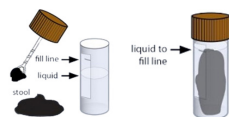
5. Using the plastic spoon from the container cup (a), place 2 to 3 small scoopfuls of stool into each specimen cup, until liquid reaches the fill line. Screw the container's cap tight enough so the liquid remains in the container and doesn't overflow or drip.

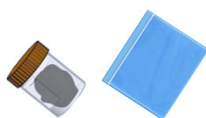6. Once specimen collection finished, place it into the plastic biohazard envelope (c), and put it into the packaging box (e)

7. Please fill in all the necessary information in the enclosed form (d). Put the form inside the packaging box and return it to the laboratory along with the STOOL specimen container

➔ STEP BY STEP INSTRUCTIONS:

1. Place the content of the test in the bathroom where you can reach them easily. Go for pee prior use.

2. Wash your hands meticulously with warm water and soap, at least 20 seconds. Dry your hands with a paper towel and use that same towel to turn off the faucet.

3. Prepare the toilet. Lift the toilet seat. Place the collection bag (b) on the toilet bowl rim. Close the toilet seat on the collection bag. This will hold the collection bag in place. Put some toilet paper in bottom of bag

4. Sit on the toilet to have a bowel movement (poop) as usual. Don't pee on your stool or in the collection bag, just pass a stool into the bag

➔ **USE THE SMALL BOX** of your KIT to package your Sampling specimens and send it back to our Lab.

ANALYSIS LAB

Tsimiski 51, PC: 54623,

THESSALONIKI, GREECE

Tel: +30 2310 256388, +30 2310 227089

E-mail: analysi@analysi.gr

➔ WAYS OF SENDING YOUR SAMPLING

- As soon as you execute correctly the whole procedure, come in contact with us- STRICTLY within 24h - in order to sent our courier at the time and place best suits you to collect your specimens.

SEND US using one of the following

E-Mail : info@ismadeofnature.com

Viber :+306934831964

What's up : +306934831964

Your : Name, Address, Mobile Phone and also the day and hour best suits you to book an appointment with our courier partner

**BE SURE WE HAVE RECEIVE YOUR
DETAILS BY RESPONDING
TO YOUR MESSAGE**

Thank you!

I-M-O-N METHODOS

<https://studio.youtube.com/video/YIvJm9m9ro0/edit>

<https://imonmed.com/services/labtests/>