I-M-O-N METHODOS

EL

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I-M-Ó-N METHODOS

Lab Test I-M-Ó-N KIT

Dry Urine Spot (DUS) Sampling Instructions

A. Content of DUS Lab test IMON KIT

- Paper filter strips (DUS strips) (a). 
- Micro Collector for urine (b). 
- Plastic glove (c). 
- Plastic envelope for biohazard (d). 
- Information form & package kit (e). 

B. STEP BY STEP INSTRUCTIONS

1. Wash your hands meticulously with warm water and soap. 

2. Go for pee and take with you the urine micro collector (b). Before urine, wipe your genital with sanitizing towelettes to prevent contamination.

3. Put the plastic glove (c). Leave the first part of the urine void (15-30ml) into the toilet to wash away any bacteria colonising the distal end of the urethra. Obtain the midstream urine specimen into the micro collector (b).

4. Place the paper strip (DUS strip) (a) up to the dotted line, inside the micro urine collector and hold for 20-30 seconds.


5. Lift the DUS strip and hold it in the air for 30 seconds (so that the excess drops of urine fall).

6. Repeat STEPS 4 & 5, in order to collect urine samples with all the DUS strips that there are inside the IMON KIT.


7. Leave the DUS strip in the air for at least 6 hours until it dries completely. You can use the second micro-collector or a plastic cup as a basis. Place it in a well-ventilated place but carefully: don't drop down the filter paper or come in contact with dust or dirt. ATTENTION!!! This STEP may demand more than 6 hours. The strip should not be folded and enclosed into plastic envelope until it is completely dry. Please before sent the samples makes sure the strips are completely DRY!!!! In case the samples keep humidity and placed to the envelope, they are NOT VALIDATED!!!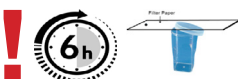


Trying to track the time in your head is too imprecise. Set a timer or keep a close eye on the second hand of your watch so πότε έχει ολοκληρωθεί η δοκιμή.

8. Final check in order to get ready the specimens for posting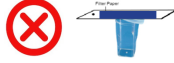

Wet and Moist, not ready to post it

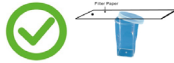Completely dry, ready to post it

9. Please enclose the DUS sticks into the plastic biohazard envelope (d). And put it into the packaging box (f)

10. Please fill in all the necessary information in the enclosed form (e). Put the form inside the packaging box and return it to the laboratory along with the specimen paper filter sticks

USE THE SMALL BOX of your KIT to package your Sampling specimens and send it back to our Lab.

ANALYSIS LAB

Tsimiski 51, PC: 54623,

THESSALONIKI, GREECE

Tel: +30 2310 256388, +30 2310 227089

E-mail: analysis@analysis.gr

WAYS OF SENDING YOUR SAMPLING

- As soon as you execute correctly the whole procedure, come in contact with us- STRICTLY within 24h - in order to sent our courier at the time and place best suits you to collect your specimens.

SEND US using one of the following

E-Mail : info@ismadeofnature.com

Viber : +306934831964

What's up : +306934831964

Your : Name, Address, Mobile Phone and also the day and hour best suits you to book an appointment with our courier partner

**BE SURE WE HAVE RECEIVE YOUR
DETAILS BY RESPONDING
TO YOUR MESSAGE**

Thank you!

I-M-Ó-N METHODOS

<https://www.youtube.com/watch?v=c28nYVdtZJI>

=c28nYVdtZJI

<https://imonmed.com/services/labtests/>